

MANUAL CLEANING

# ORANGE SOLVENT



SPECIAL CLEANER FOR SURFACE CLEANING OF MEDICAL DEVICES

1 / 3

With natural orange oil

With extra cleaning power

Effortlessly removes sticky residues and markings



MANUAL CLEANING

# ORANGE SOLVENT



## PRODUCT DESCRIPTION

2 / 3

**ORANGE SOLVENT** is a ready-to-use cleaning solution for thorough removal of stubborn residues on medical instruments and equipment.

Also for removal of adhesive residue, permanent marker and scuff marks on flooring. Economic to use.

## APPLICATIONS AND NOTES

### Range of application

For thorough removal of stubborn residues on medical instruments and equipment. Also for removal of adhesive residue, permanent marker and scuff marks on flooring.

### Application

Apply ORANGE SOLVENT to a wipe or single-use cloth and rub the area to be treated.

According to the EU Medical Device Regulation, users/patients are obligated to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the EU Member State in which the user/patient is established.

### Application notes

ORANGE SOLVENT must not be mixed with other process chemicals.  
Easy removal of stubborn stickers and old labels after a short application time and easy removal of adhesive residue.

- For substantial removal of iodine specks and other colour signal specks.
- For removing stubborn scuff marks on PVC and linoleum flooring such as those caused by bed castors.
- Dissolves greasy stains on clothing, carpets and cushions.

### Composition

Citrus Sinensis Peel Extract, Limonene, Decyl Oleate.

### Material compatibility

The use is not recommended for the following materials: polycarbonate (PC), polymethylmethacrylate (PMMA). Use is only recommended to a limited extent for disposable plastic cups, disposable latex gloves and the following materials: Acrylonitrile butadiene styrene (ABS), polyoxymethylene (POM).

### Precautionary and hazard statements

Flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May be fatal if swallowed and enters airways. Toxic to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Immediately call a POISON CENTER/doctor. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. If skin irritation or rash occurs: Get medical advice/attention. Avoid release to the environment. Dispose of contents/container to approved disposal company or local collection.

For professional use only by personnel with corresponding specialist knowledge according to national directives.

### Read label before use.

MANUAL CLEANING

# ORANGE SOLVENT



## PURCHASING INFORMATION

3 / 3

Product	Single unit	Delivery unit	Content	REF
ORANGE SOLVENT	bottle	9	250 mL	00-805-0025
	bottle	6	500 mL	00-805-005

National information may differ. For further information, please contact our subsidiary or your local dealer.  
The availability of the products and container sizes depend on a completed national registration.



## CERTIFICATIONS



For an overview of our current certifications, scan the QR code to visit:  
<https://www.schumacher-online.com/en/cert>

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO 9001, DIN EN ISO 14001, BS OHSAS 18001,  
has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH.