

SURFACE / READY TO USE DISINFECTANTS

## DESCOCEPT SPEZIAL WIPES



PREVENT

ALCOHOL-FREE WIPES FOR RAPID DISINFECTION

1 / 8

**Universal, comfortable and timesaving  
application** in a usual dispenser bucket

Basic, **rapid efficacy**

Especially suited for disinfection  
**of large surfaces in routine areas**



SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## PRODUCT DESCRIPTION

2 / 8

**DESCOCEPT SPEZIAL WIPES** are alcohol-free, ready-to-use Wipes for rapid disinfection and cleaning of medical devices and medical inventory as well as all kind of surfaces in all areas of the routine.

**DESCOCEPT SPEZIAL WIPES** offer a universal and time saving application for the disinfection and cleaning of large surfaces. The wipes can be extracted indi-

vidually from the wipes dispenser system with the re-sealable lid. Each refill pack comes with an ULTRASOL OXY SACHET to ensure proper, manual reprocessing before refilling the dispenser.

## APPLICATIONS AND NOTES

### According to Biocidal Products Regulation (BPR)

For rapid disinfection and cleaning of alcohol sensitive medical equipment and all kind of surfaces.

### Acc. to EU Medical Devices Regulation

For rapid disinfection and cleaning of alcohol sensitive, non invasive medical devices.

### Application

Wipe surfaces with DESCOCEPT SPEZIAL WIPES until completely wet and allow to dry for the entire contact time.

Medical devices that come into contact with the skin during intended use should be rinsed for 15 s under running water of suitable quality after the exposure time has expired.

According to the EU Medical Device Regulation, users/patients are obligated to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the EU Member State in which the user/patient is established.

### Application notes

Shelf life after opening: 28 days.

### Composition

100 g solution contain: 0.5 g alkyl(C<sub>12-16</sub>)dimethylbenzylammonium chloride, 0.25 g didecyltrimethylammonium chloride.

### Material compatibility

Wide range of applications on surfaces and medical devices. (see page 4 – 7)

### Precautionary and hazard statements

Harmful to aquatic life with long lasting effects. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Avoid release to the environment. Dispose of contents/ container to approved disposal company or local collection.

For professional use only by personnel with corresponding specialist knowledge according to national directives.

### Use disinfectants safely.

**Always read label and product information before use.**

SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## SPECTRUM OF EFFICACY AND CONTACT TIMES

3 / 8



SPECTRUM OF EFFICACY AND CONTACT TIMES			1 min	2 min
application recommendation for surface disinfection				
bactericidal <sup>1</sup> , yeasticidal <sup>1</sup>	VAH/EN <sup>2</sup>	with mechanical action, dirty conditions	•	
active against enveloped viruses	EN 14476	clean and dirty conditions	•	
additional test results				
bactericidal	EN 16615	dirty conditions	•	
	EN 13727	dirty conditions	•	
	EN 1276	dirty conditions	•	
yeasticidal (C. albicans)	EN 16615	dirty conditions	•	
	EN 13624	dirty conditions	•	
	EN 1650	dirty conditions	•	
active against papova/SV40 viruses	EN 14476	dirty conditions		•

<sup>1</sup> – including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)

<sup>2</sup> – EN 13624, EN 13727, EN 16615

SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## MATERIAL COMPATIBILITY

4 / 8



MATERIAL METALS	not recommended	limited recommended	recommended	APPLICATION PRODUCT EXAMPLE
stainless steel V2A			<ul style="list-style-type: none"><li>●</li></ul>	Medical transport chairs
			<ul style="list-style-type: none"><li>●</li></ul>	Transport trolleys
			<ul style="list-style-type: none"><li>●</li></ul>	rollator
aluminum			<ul style="list-style-type: none"><li>●</li></ul>	
brass	●			Stethoscope (chrome-plated brass)
copper	●			

SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## MATERIAL COMPATIBILITY

5 / 8



MATERIAL PLASTICS: ELASTOMERS	not recommended	limited recommended	recommended	APPLICATION PRODUCT EXAMPLE
PUR (polyurethane)			<ul style="list-style-type: none"><li>●</li></ul>	Medical transport chairs
			<ul style="list-style-type: none"><li>●</li></ul>	Dashboards
silicones			<ul style="list-style-type: none"><li>●</li></ul>	Face masks
			<ul style="list-style-type: none"><li>●</li></ul>	Open cuff face mask
			<ul style="list-style-type: none"><li>●</li></ul>	Medical keyboards and mice
EPDM (ethylene propylene diene (monomer) rubber)			<ul style="list-style-type: none"><li>●</li></ul>	Nursing trolleys
CR (neoprene)			<ul style="list-style-type: none"><li>●</li></ul>	

SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## MATERIAL COMPATIBILITY

6 / 8



MATERIAL PLASTICS: THERMOPLASTICS	not recommended	limited recommended	recommended	APPLICATION PRODUCT EXAMPLE
PA (polyamide)			•	X-ray apron
PE-HD (polyethylene-High Density)			•	Storage and transport containers
PVC (polyvinylchloride)			•	
PMMA (polymethylmethacrylate)			•	Acrylic and plexiglass, incubators
POM (polyoxymethylene)			•	Plastic chairs
PSU (polysulfone)			•	
EPP (expanded (foamed) polypropylene)			•	Fascia rollers
PC (polycarbonate e.g. Makrolon)	•			Syringe drivers
	•			Ultrasound devices
ABS (acrylonitril-butadiene-styrene)	•			X-ray equipment
	•			Patient monitoring monitors

SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## MATERIAL COMPATIBILITY

7 / 8



MATERIAL TEXTILES				APPLICATION PRODUCT EXAMPLE
	not recommended	limited recommended	recommended	
synthetic leather (e.g. with PVC and PUR top layer)				• Examination and treatment couches
				• Positioning aid
				• Gynaecological examination chair
				• Dental chairs
				• Operating tables
				• Patient transport couches

SURFACE / READY TO USE DISINFECTANTS

# DESCOCEPT SPEZIAL WIPES



PREVENT

## PURCHASING INFORMATION

8 / 8

Product	Single Unit	Delivery Unit	Content	Wipe Size	Art. no.
DESCOCEPT SPEZIAL WIPES	Non-woven wipe dispenser	1	100 wipes	30 x 32 cm	00-208-ERS100E
	refill pack	2	100 wipes	30 x 32 cm	00-208-RS100E
	wall mount	1	-	-	00-902-EIM

The availability of the products and container sizes depends on a completed national registration.

pH 0 1 2 3 4 5 6 7 8 9 10 11 12 13 14  0123

## CERTIFICATIONS



For an overview of our current certifications, scan the QR code to visit:  
<https://www.schumacher-online.com/en/cert>

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO 9001, DIN EN ISO 14001, BS OHSAS 18001,  
has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH.