

Operating instructions

Disinfectant and liquidsoap dispenser SPE 500/SPE 1000 with lever and disposable pump

Suitable for bottles made by all the major disinfectant manufacturers.

Before assembly:

Unpack the box, remove the plastic packaging around the dispenser and make sure that the following parts are included:


- 1 dispenser with mounted plastic disposable pump
- 1 lever to be mounted on the dispenser
- 1 wall plate for wall attachment (not necessary if there is already a suitable wall plate available and mounted)
- 1 drip tray (for insertion after the dispenser has been mounted)
- 1 bag with a set of three screws and dowels as well as a metal stripe
- 1 operating manual in German and English

Note: Determine the position of the dispenser before mounting it on the wall

- Attach above a washbasin if required (not necessary because of the integrated drip tray)
- Mounting height above a washbasin: ca. 15 to 20 cm (bottom edge of dispenser)
- Mounting height (floor to bottom edge of dispenser) on the wall: approx. 120 cm
- If dispensers are mounted in corridors: Choose the mounting height in such a way that the dispensers cannot become damaged by ward trolleys or similar


Single Unit	Size	REF
SPE 500/1000		
wall dispenser	500 ml	00-918-005
wall dispenser	1 L	00-918-010
ESH adapter for SPE 500 size	500 ml	00-918-KOMB-01

Complete assembly of the dispenser

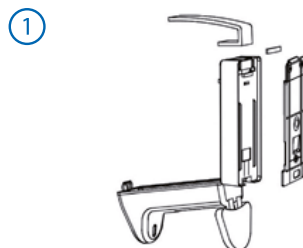

For reasons of space, the dispenser is delivered with the lever removed. To assemble it on the back of the dispenser, insert the lever and press until you hear it lock in place; replace the metal strip (is used to secure the lever, and at the same time as the upper attachment point for the dispenser on the wall plate)

Mount dispenser

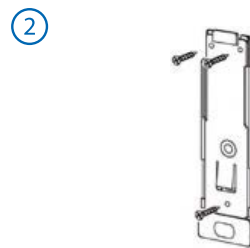

Attach the wall plate using the dowels and screws, (not necessary if a wall plate is already available). Push the dispenser into the wall plate from the top until you hear the locking pin lock in place.

Initial use




Open the dispenser, to do this press the button at the top underneath the lever (Fig. 2) and fold the cover down forwards. Push the locking pin of the wall plate out through the recess in the back of the dispenser, pull the wall plate down and out of the housing



Push a full bottle (disinfectant or soap) over the dispenser pump from below and insert it in the housing; close the housing cover and insert the drip dish. The first time you use the dispenser, press the lever down as far as it will go several times until liquid comes out of the pump nozzle, collect this in a suitable receptacle.

Dosing unit

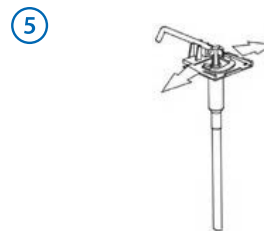

On delivery, the dispenser pump is set to the maximum dosing quantity of 1.8 ml per stroke. This amount can be reduced to 1.2 ml or 0.8 ml per stroke by moving the dosing fork to the right or left.

Cleaning the dispenser / replacing the pump

The **dispenser** is basically maintenance-free, but should be cleaned and disinfected regularly if appropriate on the basis of infection prevention and hygiene recommendations. Only use a mild detergent to wipe down the **housing**, treat with disinfectant afterwards if required. The housing can also be cleaned in a suitable dishwasher. We do not recommend cleaning the pump in order to prevent hygiene problems arising: The plastic pumps have been designed as disposable pumps and can be purchased at a favourable price - maybe at even less cost that would be incurred for cleaning - as a spare part. For this reason, the pumps should be replaced if necessary, and the used pumps should be disposed of properly.

